

Product details
SpeediCath® Flex

Catheter length	FR	Catheters per box	Product code
17" / 44 cm	10	30	28920
17" / 44 cm	12	30	28922
17" / 44 cm	14	30	28924
17" / 44 cm	16	30	28916

Experience an easier catheterization.
Try SpeediCath Flex.

Call **1-866-293-6349**
or go to coloplast.ca/SpeediCathFlex
for free samples and coverage support

Coloplast develops products and services that make life easier for people with very personal and private medical conditions. Working closely with the people who use our products, we create solutions that are sensitive to their special needs. We call this intimate healthcare.

Our business includes ostomy care, continence care, wound and skin care and urology care. We operate globally and employ more than 10,000 employees.

Important Safety Information: SpeediCath® Flex is indicated for use by patients with chronic urine retention and patients with a post void residual volume (PVR) due to neurogenic and non-neurogenic voiding dysfunction. The catheter is inserted into the urethra to reach the bladder allowing urine to drain. This device is intended for males only. Patients performing self-catheterization should follow the advice of, and direct questions about use of the product to, their medical professional. Before using the device, carefully read the product labels and information accompanying the device including the instructions for use which contain additional safety information. The SpeediCath catheter is for single-use only; discard it after use. If you experience symptoms of a urinary tract infection, or are unable to pass the catheter into the bladder, contact your healthcare professional. The risk information provided here is not comprehensive. To learn more, talk to your healthcare provider. For further informations, call Coloplast Canada Corporation at 1-877-820-7008 and/or consult the company website at www.coloplast.ca.

[Ostomy Care](#) / [Continence Care](#) / [Wound & Skin Care](#) / [Urology Care](#)

Coloplast Canada Corporation, Mississauga, Ontario
www.coloplast.ca The Coloplast logo is a registered trademark of Coloplast A/S.
© 2016-10 All rights reserved



E0464N (1500)

Introducing
SpeediCath® Flex



SpeediCath® Flex

Every detail makes it
easy

SpeediCath® Flex is the new soft catheter with a dry-sleeve and a flexible tip. It has a range of features that makes every step of catheterization easy and is designed for use in normal and more complex male anatomies.

SpeediCath Flex is:

- Hygienic**
- Flexible**
- Discreet**

Package size	10 x 23 cm / 4 x 9"
Catheter length	44 cm / 17" from drainage end to catheter tip
Available in diameters	FR 10, 12, 14 & 16
Coating	The renowned SpeediCath® hydrophilic coating
Urine bag	Connection through drainage end

There are many
special reasons
 why SpeediCath® Flex
 makes catheterization easy

Hygienic




1. Soft-squeeze grip

The grip enables positioning of the catheter tip for safe guidance into the urethra and control during catheterization – all without touching the tip in any way. Softly squeezing the grip pauses the movement of the catheter.

2. Dry-sleeve

The sleeve enables insertion of the catheter without touching the catheter tube. While the catheter has the unique SpeediCath® hydrophilic coating for smooth passage, the sleeve is dry so it can be gripped and handled cleanly.

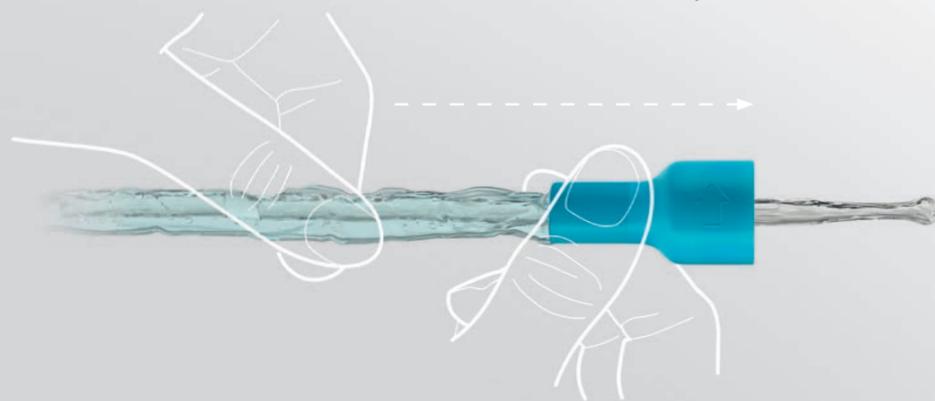

Flexible



3. Soft catheter

The softness of the catheter tube makes it easy to handle and enables safe and gentle passage through your urethra.

4. Flexible tip

The soft bead at the end of the tip centres the catheter in the urethra for smooth insertion. The flexibility of the top section enables easy guidance through curves and bends.

Discreet



5. Re-closable

After use the protective grip and drainage end can be joined together again. This enables tidy and hygienic handling by keeping any residual liquid locked within the catheter.

6. Practical packaging

The packaging is easy to open and has a neutral non-medical look. After use, the re-closed catheter can be returned to the packaging. The packaging can then be closed and folded for discreet disposal.

